**Xblitz**®

## Spirit 2


## Table of contents

### Spis treści

<b>4</b>	<b>Instrukcja obsługi</b>	<b>PL</b>
<b>8</b>	<b>User manual</b>	<b>EN</b>
<b>12</b>	<b>Bedienungsanleitung</b>	<b>DE</b>
<b>16</b>	<b>Návod k použití</b>	<b>CZ</b>
<b>20</b>	<b>Használati utasítás</b>	<b>HU</b>
<b>24</b>	<b>Používateľská príručka</b>	<b>SK</b>
<b>28</b>	<b>Vartotojo vadovas</b>	<b>LT</b>
<b>32</b>	<b>Lietotāja rokasgrāmata</b>	<b>LV</b>
<b>36</b>	<b>Kasutusjuhend</b>	<b>ET</b>
<b>40</b>	<b>Manual de utilizare</b>	<b>RO</b>
<b>44</b>	<b>Ръководство за потребителя</b>	<b>BG</b>
<b>49</b>	<b>Manuel d'instructions</b>	<b>FR</b>
<b>53</b>	<b>Manuale di istruzioni</b>	<b>IT</b>
<b>57</b>	<b>Manual del usuario</b>	<b>ES</b>

**A**

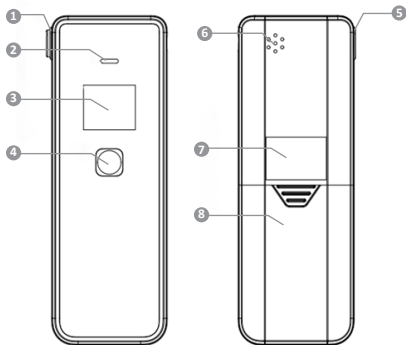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## USER MANUAL

Dear Customer,

Thank you for choosing the Xblitz Spirit 2. The device you have purchased is a semiconductor breathalyser for measuring blood alcohol content from exhaled air. Please read the instructions below before use.

**CAUTION:** Do not allow yourself or anyone else to drive after drinking alcohol.

### 01. CONTENTS OF THE KIT

1. Breathalyser
2. AA battery x2
3. Mouthpiece x5
4. Operating instructions

### 02. DESCRIPTION OF THE DEVICE

SEE FIG. A

1. Mouthpiece hole
2. Indicator light
3. Display
4. Button
5. Air outlet
6. Ventilation opening
7. Label
8. Battery cover

### 03. DESCRIPTION OF THE DISPLAY

SEE FIG. B

1. Battery indicator
2. Test result, messages
3. Unit

#### 04. DISPLAY

The following are the messages that may appear on the display and their meaning.

1. **Message:** „C”  
**Meaning:** Gas detected, please continue to blow
2. **Message:** „-”  
**Meaning:** Analysis in progress, please wait
3. **Message:** „Err”  
**Meaning:** Sensor malfunction
4. **Message:** „rP”  
**Meaning:** Unstable exhalation or interrupted exhalation
5. **Message:** „blo”  
**Meaning:** Start blowing into the mouthpiece of the device
6. **Message:** „Lob”  
**Meaning:** Low battery level

#### 05. OPERATION OF THE DEVICE

1. Remove the battery box cover on the back of the unit and place two 1.5V AA batteries inside. Make sure the battery terminals are facing the correct way and reattach the container cover.
2. Fit the mouthpiece to the side so that it is at 90 degrees to the rest of the device.
3. To start the device, hold down the button on the device for approximately 2 seconds. A beep will sound from the device, the LED will light up blue and the display will show a countdown from 20 to 0. Wait until the countdown has finished to ensure that the device sensor is ready for use.  
**NOTE:** Do not blow into the mouthpiece until the countdown is complete. Premature use of the device may lead to an incorrect measurement of the breath alcohol content.
4. When the countdown is over, the display will show “blo”. Take a deep breath and start blowing into the mouthpiece. The display will show a string of letters “C”, continue blowing until a beep sounds from the device. After a few seconds, the measurement result will appear on the display. The result of the measurement is given in per mille (blood alcohol content).  
**NOTE:** If the exhaled air flow is not constant or is interrupted during the measurement, the display will show “rP”.
5. If the measured blood alcohol level is below 0.2 ‰, the LED under the display will light up green. If the measured blood alcohol level is above 0.2 ‰, the LED will light up red and an alarm will sound from the device.
6. If you want to manually switch off the device, hold down the button for 3 seconds. The device will switch off spontaneously a few tens of seconds after the last activity.
7. To change the unit of the result after the measurement is finished, press the

key. Pressing the key switches to the next unit.

## 06. PRECAUTIONS

1. Do not drop the device or shake it violently.
2. Test results may be inaccurate when battery capacity is low.
3. To ensure the accuracy of the measurement, perform the test 15 minutes after drinking alcohol.
4. If the device has been left unused for a long time, the first measurement taken may be inaccurate.
5. If you want to perform multiple tests, wait at least 1 minute between tests.
6. Do not blow tobacco smoke, liquids or pieces of food into the device. This can permanently damage the sensors and lead to incorrect measurements.
7. If the device is dirty, use a gentle material to wipe it down. Do not use caustic cleaners or wire or hard cleaning utensils, they could damage the appliance casing. Do not use caustic cleaners or wire or hard cleaning tools, as these could damage the appliance.
8. The correlation between breath alcohol content and blood alcohol concentration depends on a number of variables, including environmental factors (such as wind, humidity, temperature, etc.) as well as the health of the person being tested.
9. The measurement results from this device do not constitute evidence in legal terms and cannot be used in court. The driver bears full responsibility for driving under the influence of alcohol.
10. The result of a test can vary depending on many factors. Getting a given test result one time does not mean that the result will be the same even a few minutes later.
11. Do not allow yourself or anyone else to drive after consuming alcohol. Always use a taxi or other vehicle driven by a sober person when you or someone else has consumed alcohol.
12. Under normal use, the device needs to be recalibrated approximately every 6 months or after 500 tests performed. For accurate measurement results, contact your device distributor or service centre and ask them to recalibrate the device's sensor.

## 07. FAQ

The following are the messages that may appear on the display and their meaning.

1. **Error:** Nothing is displayed on the screen.  
**Possible reason:** Incorrectly installed batteries  
**Solution:** Correctly insert the batteries in the device  
**Possible reason:** Batteries close to discharge  
**Solution:** Replace the batteries  
**Possible cause:** Circuit failure

- Solution:** Contact an authorised service centre
2. **Error:** No response to air being blown in  
**Possible cause:** Unheated sensor  
**Solution:** Wait until the countdown on the display is over.  
**Possible cause:** Circuit failure  
**Solution:** Contact an authorised service centre
3. **Error:** "Err" message appears and the device switches off  
**Possible cause:** Sensor malfunction  
**Solution:** Contact an authorised service centre
4. **Error:** Unable to switch on the device  
**Possible reason:** Discharged batteries  
**Solution:** Replace the batteries

## 08. SPECIFICATION

**Sensor type:** Semiconductor

**Measurement range:** 0~2.00‰ BAC (0~1.00mg/L; 0~2.00g/L; 0~0.20% BAC; 0~1.00mg/L).

**Measurement resolution:** 0.01BAC‰

**Standby time:** Approximately 35 s

**Operating temperature:** 0~50 °C

**Permissible air humidity:** ≤ 93 %

**Display:** LCD

**Batteries:** 2x 1.5V AA

**Dimensions:** 124x44x19 mm

**Weight:** 49g (without battery)

**Error range:** ± 0.1‰ BAC for 0.5BAC‰

## 09. WARRANTY CARD

The product comes with a 60-month warranty. Warranty conditions can be found at: <https://xblitz.pl/gwarancja>

Complaints should be made using the complaint form found at: <https://reklamacje.kgktrend.pl/>

Details, contact and address of the service can be found at: [www.xblitz.pl](http://www.xblitz.pl)

Serial number.....

The guarantee is only valid if the device is regularly serviced and calibrated at an authorised service centre at least once a year. The device has been correctly calibrated since new. Every breathalyser offered on the market requires periodic calibration, i.e. every 12 months (every 6 months is recommended), the device must be configured so that it correctly interprets the amount of alcohol in the air exhaled. The need for recalibration is not a defect in the device and is not subject to warranty. A device that is not regularly calibrated will show erroneous results.

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EN

The crossed out trash can symbol indicates that unusable electrical or electronic devices, its accessories (such as power supplies, cords) or components (for example batteries, if included) cannot be disposed of alongside with household waste. In order to dispose of the devices or its components (for example, batteries) deliver the device to the collection point, where it will be accepted free of charge. Disposal is subject to the recast version of the WEEE Directive (2012/19/ EU) and the Directive on batteries and accumulators (2006/66 / EC). Proper disposal of the device prevents degradation of the natural environment. Information about the collection points of the facilities is issued by the competent local authorities. Incorrect disposal of waste is subject to penalties provided for by the law in force in the given area.

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